INHEALTH

TECHNOLOGIES®

INSTRUCTIONS FOR USE

Blom-Singer® Electrolarynx Digital EL 2000

PACKAGE CONTENTS


- 1x EL 2000 Digital Electronic Larynx Device
- 1x Magnetic USB Charging Cable
- 1x Safety Lanyard
- 1x User Manual

TABLE OF CONTENTS

EN	ENGLISH	3
15	عربي	AR
DE	DEUTSCH	28
FR	FRANÇAIS	41
54	עברית	HE
IT	ITALIANO	67
PT	PORTUGUÊS	80
ES	ESPAÑOL	93
TR	TÜRKÇE	106

PARTS AND FEATURES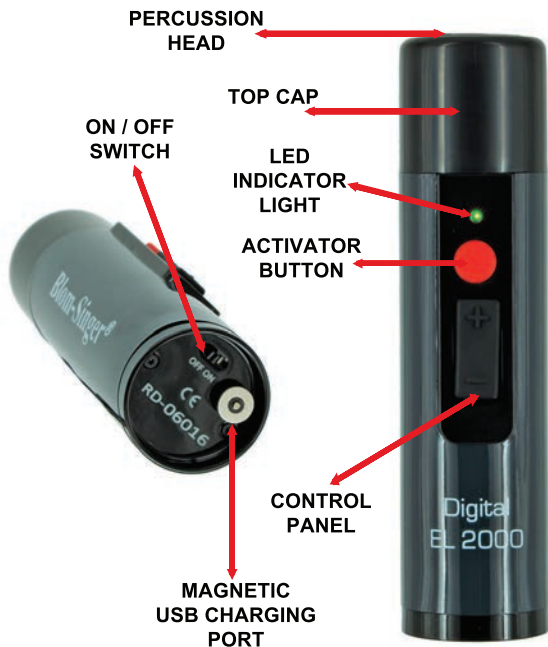

INTENDED USE

An electrolarynx is a battery-powered artificial larynx that is externally applied and intended for use in the absence of the larynx or the inability to use the larynx to produce sound. When held against the skin in the area of the voice box, or by insertion of a tube in the oral cavity (with an oral adapter), the device generates mechanical vibrations which resonate in the oral and nasal cavities and can be modulated by the tongue and lips in a normal manner, thereby allowing the production of speech.

CONTRAINDICATIONS

The device should only be used in accordance with the Instructions for Use. Users without the physical, cognitive, or mental ability required to operate the device themselves, should not use the device independently and should only use it if they are under sufficient supervision of a clinician or a trained caregiver. The device should not be directly applied over frail neck tissue with weak blood vessels. This can cause tissue damage or bleeding. Patients with this condition should only use the device when they have been specifically instructed by their clinician about how to use the device and where to safely apply it.

DESCRIPTION OF THE DEVICE

An electrolarynx is a battery-powered artificial larynx that is externally applied on undamaged skin and intended for use in the absence of the larynx or the inability to use the larynx to produce sound.

WARNINGS

Failure to follow the safety instructions could result in personal injury, or damage the device. Read all safety information below before using the device.

Incorrect use of the device or not following instructions in this IFU can cause personal injury and/or damage the device.

- Do not use the device between persons, this can cause cross contamination. The device is intended for single patient use only.
- Do not turn on, or use, or charge the device if it has been exposed or dropped in water or other liquids, as it can cause personal injury and/or damage the device.
- Before charging inspect the charger for damage. A damaged or non functioning charger shall no longer be used, as it can cause fire, personal injury or damage the device.
- Only use approved chargers, as usage of unapproved chargers can cause fire, personal injury, or damage to the device.
- Do not expose the lithium battery to temperatures outside the temperature ranges specified on the label, as it could result in personal injury or damage the device.
- The device contains a magnet that generates magnetic and electromagnetic fields that may interfere with pacemakers or other implantable devices as well as certain procedures or treatments. Maintain a minimum distance of 6"/15 cm between the device and any medically implanted devices. Consult with your physician before any medical procedure or treatment. If interference between the device and any medically implanted device is suspected, discontinue use and consult with your physician.
- Do not insert any part of the device or accessories into the stoma, as this could result in personal injury.
- Do not modify the device, as it could result in personal injury or damage the device.
- The device contains small parts that may become dislodged and present a choking hazard. Keep away from small children.

PRECAUTIONS

- Be careful not to damage the magnetic charging port when operating or charging the device.
- Failure to follow maintenance instructions could result in personal injury and/or damage the device.
- Never use a broken, modified or altered device; or charge a broken, modified or altered device, as this could result in personal injury and/or damage the device.
- The device and batteries may become warm (up to 45 °C/ 13 °F) during charging. Allow the device to cool down before use.
- The device may become warm during use. Take necessary precautions when placing on neck for use.
- With extensive use, vibrations can cause discomfort for some sensitive users.
- The device may malfunction or stop working due to electro-magnetic field interference from other devices.
- Be careful not to chew/bite on the oral tube as it could cause dental damages.
- Be careful not to drop the device, as it could cause damage to the device.
- The device may cause sense of effort and fatigue in the shoulder/arm for individuals who experiences shoulder dysfunction.

GETTING STARTED

BEFORE FIRST USE, FULLY CHARGE THE DEVICE

Charging the Device

- 1 Connect the included USB charging cable (Fig. 2) to the magnetic charging port on the device (Fig. 1).

WARNING: DO NOT REMOVE THE MAGNETIC USB PORT

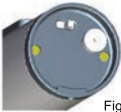
Fig. 1

- 2 Connect the charging cable to a powered USB port (Fig. 3), such as the included USB charger adapter, power bank, car charger, or computer.

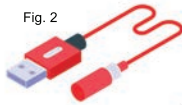
Fig. 2

Fig. 3

NOTE: THE DEVICE WILL NOT OPERATE WHILE CHARGING

- 3 A red LED indicator will slowly blink when charging (Fig. 4). Continue charging the device until the LED Indicator Light turns solid green (Fig. 5).

Fig. 4

Blinking Red LED Indicator Light = Charging

Fig. 5


Solid Green LED Indicator Light = Fully Charged

- 4 Once the LED Indicator Light Turns green, remove the charging cable. The device is now fully charged and ready for use. Your device will last up to 30 hours on one charge.

OPERATING THE DEVICE

Turning the Device ON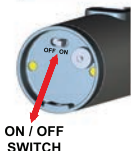


To turn the device on, gently slide the power switch located on the bottom of the device to the ON position. The device will boot up and run a self-diagnosis that will last approximately 3 seconds. During the system bootup sequence, the Green LED Indicator Light will be solid green. The LED Indicator Light will turn off when the device is ready for use.

NOTE: CHARGE THE DEVICE IF THE LED INDICATOR LIGHT IS RED

Programming the Pitch

To enter PROGRAMMING MODE, press the Red Activator Button and the (-) Volume button simultaneously for one second and then release them both.

If successful, the device will start continuously buzzing. If the device does not enter programming mode, repeat the process until successful.

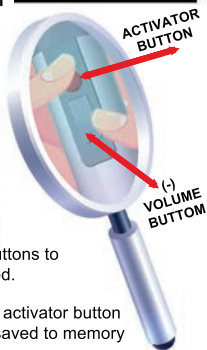Once in programming mode, use the (+) or (-) buttons to adjust the audible pitch higher or lower as needed.

To exit PROGRAMMING MODE, press the RED activator button after setting the desired pitch. This pitch will be saved to memory and become the default pitch for every use.

Adjusting the Volume

The electronic larynx device features variable volume settings. Volume level can be adjusted at any time during operation. To adjust the volume, press the (+) or (-) button as needed. The device's LED will flash RED when you reach the maximum or minimum volume setting.

TECHNICAL TROUBLESHOOTING

If the electronic larynx does not turn on, works intermittently, or if performance decreases:

- Charge the device until the red LED indicator light turns green.

If the electronic larynx does not charge:

- Inspect the device and charging cable for any damage or corrosion.
- Check the USB charging cable connections and the power source.
- Replace the USB charging cable.

IF THE PROBLEM PERSISTS, CONTACT INHEALTH FOR A REPLACEMENT

MAINTENANCE

Maintenance are the responsibility of the user and should be performed after each use.

- Wipe the device with a clean, dry cloth or if necessary, a slightly damp cloth. Use water with mild soap or 50% isopropyl alcohol.
- Be careful not to get moisture inside the device.
- Do not use electrical, spray cleaners, or harsh chemicals.

LEARNING TO SPEAK

Finding the “Sweet Spot”

The sweet spot is a location on your neck where the vibrations from the electrolarynx transmit through your neck tissue into your oral cavity with the most clarity. The best way to find your sweet spot is to firmly press the electrolarynx against different areas of your neck, activating the sound by moving your mouth and using your tongue to articulate words. Ensure an airtight seal between the sound head and your neck. Your goal is to identify the area on your neck that transmits sound most effectively. While searching for a sweet spot, focus on transmitting sound through your neck tissue and out of your oral cavity (mouth). Take your time in this process, as it is a crucial step in learning how to use an electrolarynx effectively. The ideal spot varies for each patient.

First Words

After locating your 'sweet spot,' you can begin making basic sounds. Start with single vowels, over-articulating each sound with your mouth until you become comfortable with all the vowels. Then, progress to practicing short words. Gradually work your way up to sentences and eventually, full conversation.

Try different positions until you get the best sound. Note that even a small position change can have a great impact on the sound volume.

COMMON ISSUES

Sound is not clear, muffled, or has too much buzzing

- Make sure the percussion head is in full contact with the skin on the neck.
- Try different positions on the neck until you get the best sound. Note that even a small position change can have a great impact on the sound volume.
- Reduce the volume. Adjusting the base tone up or down may produce a better voice.
- Adjust the pitch of the device. Adjusting the selected pitch up or down may produce a clearer sound.

If you are unable to achieve sound transmission through your neck or are unable to place the device against your neck for medical reasons, try cheek placement or use the optional oral adapter.

Oral adapter (sold separately) attaches to the percussion head of the electronic larynx device.

EL 2002 Oral Adapter

EXPECTATIONS

Learning to speak with an electrolarynx device is not easy. Think of it as learning to play a musical instrument. It requires time, practice, and patience. In some cases, users may achieve effective speech within a month, while in others, it can take as little as three days.

TECHNICAL DATA

Physical

Speech Aid:	L 123 x Ø 32 mm (L 4.84 in x Ø 1.26 in)
Weight:	4.5 oz (127.6 g)
Body Material:	ABS Plastic

General

Expected Service Life:	1-5 years
Frequency Range:	70 - 165 Hz
Volume Range:	60 - 83 dB
Speech Time:	Up to 30 hrs
Connector:	Magnetic Micro USB
USB Charging:	Battery Charging Spec. v1.2
Input AC:	100-240V ~ 50-60 Hz

Battery

Capacity:	3400 mAh
Battery Type:	3.7V Li-Ion
Battery Model:	Custom Pack
Battery Life:	More than 3 years
Operating Temperature:	+10°C to +30°C (50°F - 86°F) (full performance) 0°C to +40°C (32°F - 104°F)(still permissible)

SYMBOLS GLOSSARY

SYMBOL	STANDARD No.	SYMBOL TITLE & REF No.	MEANING OF SYMBOL
	EN ISO 15223-1:2016	Manufacturer; 5.1.1	Indicates the medical device manufacturer, as defined in EU Directives or Regulations
	93/42/EEC EU 2017/745	CE Marking; 93/42/EEC Annex XII, EU 2017/745 Annex V	CE marking of conformity
	EN ISO 15223-1:2016	Authorized representative in the European Community; 5.1.2	Indicates the authorized representative in the European Community
	EN ISO 15223-1:2016	Batch code; 5.1.6	Indicates the manufacturer's catalogue number so that the medical device can be identified
	EN ISO 15223-1:2016	Batch code; 5.1.5	Indicates the manufacturer's batch code so that the batch or lot can be identified
	2012/19/EU, Article 12 and Annex IX	Waste and Electronic Equipment	Goods are marked with this symbol to show that they were produced after the 13 th of August 2005, and should be disposed of separately from normal household waste so that they can be recycled
	EN ISO 15223-1:2016	Caution; 5.4.4	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself
	EN ISO 15223-1:2016	Consult instructions for use; 5.4.3	Indicates the need for the user to consult the instructions for use
	EN ISO 15223-1:2016	Date of Manufacture; 5.1.3	Indicates the date when the medical device was manufactured
	EN ISO 15223-1:2016	Serial Number; 5.1.7	Indicates the manufacturer's serial number so that a specific medical device can be identified

WARRANTY AND LIMITATION OF LIABILITY

Limited Two-Year Warranty

InHealth warrants this product for two years from the date of purchase against any defects in material or workmanship. This warranty does not cover normal wear and tear, nor damage resulting from misuse or neglect. We will, at our discretion, replace or refund the purchase price of this product should we determine it to be defective.

Limitation Of Liability

Liability limited to the purchase price of this product. InHealth shall not be liable for any incidental or consequential damages or any implied warranty on this product.

INHEALTH
TECHNOLOGIES®

Distributed by:
InHealth Technologies
A Division of Freudenberg Medical, LLC
Carpinteria, CA 93013-2918 USA
www.inhealth.com

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Freudenberg Medical, LLC, parçası olan