LARYVOX® DUO BRUSH

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









INSTRUCTIONS FOR USE

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LARYVOX® Duo Brush

Größe Size	Stimmprothesen-Größe Voice prosthesis size		Art.-Nr. REF
	Länge Length in mm	Breite Width in Fr	
6	4-6	16-24	29110-06
8	7-8	16-24	29110-08
10	9-10	16-24	29110-10
12	11-13	16-24	29110-12
18	14-18	16-24	29110-18
-	4-10	20-24	29120

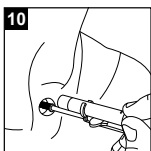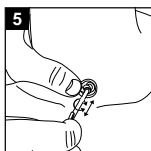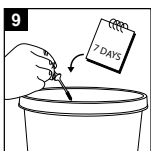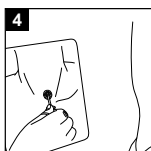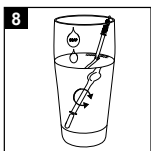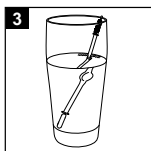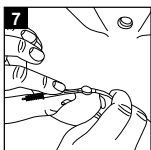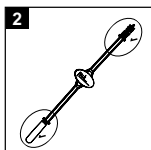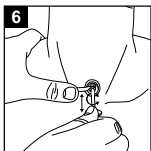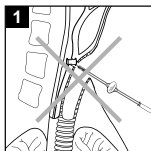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PICTROGRAM LEGEND

en

Pictogrammes listed below you'll find on the product packaging, if applicable.

Catalogue number

Batch code

Use by

Content (in pieces)

Consult instructions for use

Keep away from sunlight

Keep dry

For reuse on a single patient

Medical devices

Do not use if package is damaged

CE marking

Manufacturer

LARYVOX® DUO BRUSH

1. FOREWORD

These instructions for use are valid for the **LARYVOX® Duo Brush** cleaning brushes for voice prostheses.

These instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product for the first time!

Keep them in an easily accessible place for future reference.

Please keep the package for as long as you use the product.

It contains important information on the product!

2. SAFETY INFORMATION

This product is a single-patient product and is thus intended exclusively for one-time or repeated use in one individual patient.

The correct brush size should be selected by the doctor/physician or the trained medical professionals.

Do not perform any type of adjustment, repair or alteration on the brush or compatible accessories.

If products are damaged, they must be disposed of properly.

LARYVOX® DUO BRUSH EXTRA is not suitable for **16** French voice prostheses!

3. INTENDED PURPOSE

The cleaning brush is used for daily cleaning of the inner tube of the voice prosthesis.

4. INDICATION

Voice prosthesis

5. CONTRAINDICATION

Do not use the **LARYVOX® Duo Brush** cleaning brushes for voice prostheses for a greatly widened shunt tube, because in such a situation secure retention of the voice prosthesis in the tissue is not assured.

6. COMPLICATIONS AND ADVERSE REACTIONS

Avoid incorrect introduction of the cleaning brush with tilting/slanting effects on the voice prosthesis, since this could cause dislocation of the voice prosthesis (see Fig. 1).

7. PRODUCT DESCRIPTION

The cleaning brush has a rounded tip to prevent injuries to the oesophageal mucosa.

Secretion residues can be removed from the tracheal flange and its surroundings by means of the small sponge.

The **LARYVOX® Clip** can be used to fix a stoma lamp to the cleaning brush in order to illuminate the voice prosthesis brightly for cleaning (see Fig. 10).

The **LARYVOX® Stoma Light (REF 69101)** can be ordered separately.

8. APPLICATION

Check the **LARYVOX® Duo Brush** cleaning brushes for voice prosthesis for possible damage before each use.

Only undamaged cleaning brushes should be used for cleaning the voice prosthesis (see Fig. 2).

Moisten the brush/sponge with clean water before use (see Fig. 3).

Make sure your view is optimum (see Fig. 4).

If necessary, use the **LARYVOX® Clip** with a tracheostoma lamp in order to illuminate the voice prosthesis brightly.

Guide the cleaning brush gently into the inner tube of the voice prosthesis (see Fig. 5).

(If a replacement prosthesis is being used, it is then imperative to fix the safety thread of the voice prosthesis with at least one finger) (see Fig. 6).

Clean the voice prosthesis by gentle forward and back movements accompanied by gentle turning movements.

WARNING!

LARYVOX® DUO BRUSH EXTRA should only be used with a rotating motion, as the longer bristles can lead to dislocation of the voice prosthesis!

To optimise accessibility to the voice prosthesis, the cleaning brush may be bent individually at the flexible brush body (not at the brush wire!) (see Fig. 7).

9. HYGIENE INSTRUCTIONS

Clean the voice prosthesis at least **2** times daily with the cleaning brush.

After use, remove the residual secretions from the cleaning brush and/or the cleaning sponge with a clean cloth.

In addition, carefully rinse the cleaning brush and/or the cleaning sponge by swirling in a soap solution of liquid soap for **15** seconds (see Fig. 8).

Then rinse out with clear, almost sterile water.

Allow the cleaning brush to dry in the air.

Regular disinfection one time per day is recommended.

Especially in cases of potentially high risk of contamination (after contact with the floor or with pets, close proximity to patients with respiratory diseases or infections, and in principle in a hospital environment, where the risk of infection is high on the whole), the cleaning brush should be disinfected before as well as after each use.

A cleaning brush may be disinfected with the following substances: **70%** ethanol for **10** minutes, **70%** isopropyl alcohol for **10** minutes, **3%** hydrogen peroxide for **60** minutes.

10. STORAGE

Store in a dry environment away from sunlight and/or heat.

Store the dry cleaning brush hygienically in the associated case.

11. SERVICE LIFE

We recommend replacing the cleaning brush once a week (see Fig. 9).

12. DISPOSAL

The product must only be disposed of in accordance with the national regulations.

13. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or adverse events caused by unauthorised product alterations or improper use, care, and/or handling.

In particular, Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by modifications to the product or by repairs, if these modifications or repairs were not carried out by the manufacturer himself.

This applies both to damage caused to the products themselves as a result thereof and to all consequential damage caused thereby.

When using the product beyond the period of use specified under Section 11, and/or use, treatment, maintenance (cleaning, disinfection) or storage of the product in non-compliance with the specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects.

Should a serious incident occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, then this must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business (GTCs) which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

The manufacturer reserves the right to make changes to the product at any time without notice.

LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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