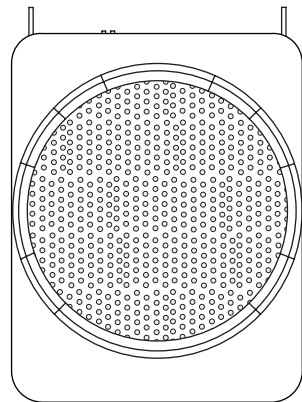
VOICETEC®

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Keys & Ports instruction

	Mic-in	In Standby mode, insert microphone to 3.5mm microphone interface to amplify.
	Aux-in	3.5mm audio interface, support music play, insert audio cable to play music automatically.
	USB-C Charging Port	Charging port supports charging through 5V mobile phone charger and computer USB port.
	Dial Volumen Switch	Rotating the dial can switch on/off the amplifier and adjust the volume. Rotate the switch clockwise (pull right) to turn on the amplifier and increase volume, rotate the switch counterclockwise (pull left) to decrease volume till turn off the power.

Operation description

1. Power switch of the voice amplifier:

Pull the dial switch clockwise to the right, hear a "click" sound, enter the power on state, the indicator light will light up, and default to Bluetooth mode.

When turned on, pull the dial counterclockwise to the left; the device is turned off once you here a "click" sound.

The indicator light will turn off.

2. Volume control of the voice amplifier:

Rotate the volume dial to adjust the volume. Pull right the button clockwise to increase the volume and pull left the button counterclockwise to decrease the volume.

3. Bluetooth connection:

The voice amplifier is in Bluetooth mode by default when turned on.

When connecting, please turn on the Bluetooth switch on your phone, find the name of the voice amplifier "S368" in the available devices, and click pairing.

You can play music or other audio files in your phone.

4. Wired microphone voice amplification:

Insert the wired microphone plug directly into the MIC interface of the voice amplifier to perform voice amplification;

When using, please wear a wired microphone first, adjust the position and angle of the microphone pickup head, and maintain a distance of about 50CM from the amplifier to avoid whistling

5. Aux mode of the voice amplifier:

Connect the voice amplifier with computers, mobile phones, PAD, MP3 , MP4 and other devices through the 3.5mm AUX audio cable to play audio, it can be used as external speaker at this time.

Charging instructions

When the battery is low, you can use the complimentary USB-C charging data cable to charge the voice amplifier through devices such as a computer, 5V phone charger, mobile power supply, etc.

The product only supports standard 5V charging and does not support all kinds of QC fast charging.

Charging with a voltage higher than 5V will burn the product or fail to charge.

Description of LED indicator lights for S368 wired voice amplifier:

Voice amplifier status		LED indicator status
Low battery		LED indicator light flashing fast
Turn on the power to charge	charging	Yellow light always on
	charged	Green light always on
Turn off the power to charge	charging	Red light always on
	charged	Green light always on

Product specification

Rated power	3W	Max power	6W
Loudspeaker	2.5 inches 2 ohmic magnetic speaker	Frequency response	200Hz-15KHz
Battery	3.7V/1800mAh, 1 lithium battery	Charging voltage	USB/DC 5V 1A
Charging time	2-3 hours	Using time	22 hours
Size	72*97*27mm	Net weight	125g

Tips and cautions

When you use it for the first time, or after a long time before you use it again, please keep the amplifier be fully charged to ensure battery life.

To avoid distortion or damage caused by excessive voltage or current, please do not insert the external sound source signal into the MIC interface.

The microphone should not be directly facing the front of the voice amplifier at close range, and it is best to face the back of the microphone. The voice amplifier itself has a function to prevent voice whistling. If the voice amplifier produces a voice whistling sound, the microphone should be pulled or moved, and the voice amplifier will immediately stop whistling.

Please don't disassemble or repair it by non-professional technical personnel. If there are any problems, please contact our company for after sale service.

To prevent the danger of short-circuiting, please keep products or accessories away from water, moisture, rain, or splash.

Do not use it in inflammable and explosive areas or in areas where radio equipment is prohibited. When you are entering dangerous areas or prohibited areas, power should be turned off to avoid potential safety hazards.

Do not use it in high temperature, high pressure, and high humidity environment (such as sunshine exposure, stove, heater, humidifier, water pool, etc.). Do not place it or close to dangerous sources (fire source, liquid, explosive, high-speed machinery, etc.) to avoid damage and danger.

To protect the environment, please put the discarded amplifier in the designated trash can or send it back to the manufacturer. It should not be discarded at will.

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Medizintechnik-Vertrieb GmbH
Shanghai Norwii Technology Co., Ltd