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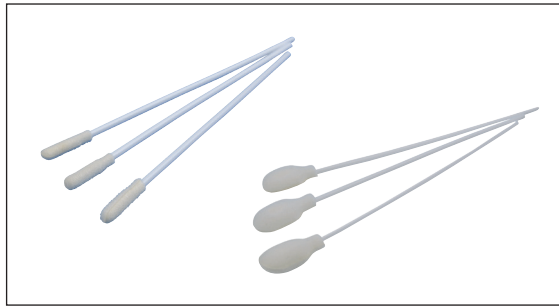
de

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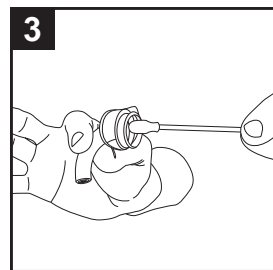
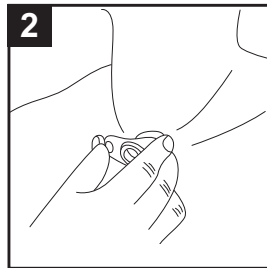
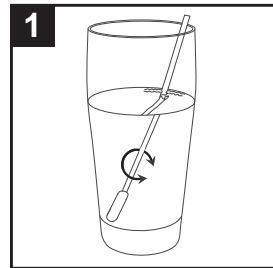
31910 OPTIBRUSH® SWAB

31920 OPTIBRUSH® SWAB XL

31925 OPTIBRUSH® SWAB XL - LENGTH 25 CM

BILDER

PIKTOGRAMM LEGENDE

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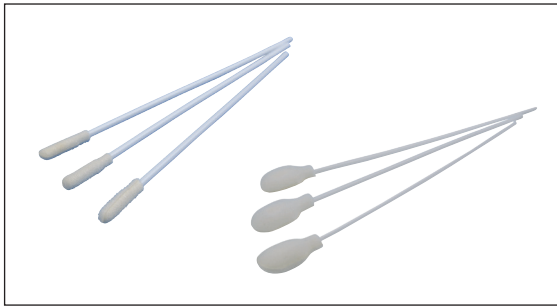
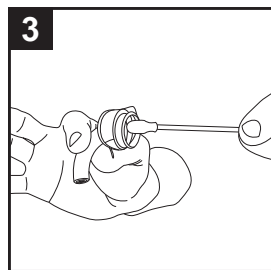
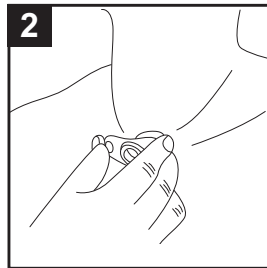
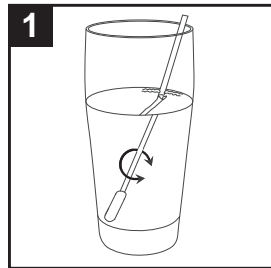
PICTURES

PICTOGRAM LEGEND

Pictogrammes listed below you'll find on the product packaging, if applicable.

	Catalogue number
	Batch code
	Use by
	Date of manufacture
	Content (in pieces)
	Consult instructions for use
	Keep away from sunlight
	Keep dry
	Do not reuse
	Medical devices
	Non-sterile
	CE marking
	Manufacturer

OPTIBRUSH® SWAB

1. FOREWORD

These instructions for use are valid for **OPTIBRUSH® SWAB**.

These instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

When used for the first time, the products must be selected, used and inserted by appropriately trained physicians or medical professionals/medical device consultants.

Please read the instructions for use carefully before using the product for the first time!

Keep them in an easily accessible place for future reference.

Please keep the package for as long as you use the product.

It contains important information on the product!

2. SAFETY INFORMATION

Never insert the cleaning swab directly into the tracheostoma!

Damaged products need to be properly disposed immediately.

OPTIBRUSH® SWAB are single-patient products and intended for single use only.

3. INTENDED PURPOSE

This non-sterile cleaning swab is intended for gentle cleaning of tracheostomy tubes.

4. INDICATION

The product is indicated for tracheotomized and laryngectomized patients.

5. CONTRAINDICATION

There are no known contraindications for this product.

6. COMPLICATIONS AND ADVERSE REACTIONS

There are no known complications or adverse reactions for this product.

7. PRODUCT DESCRIPTION

Flexible plastic swab with absorbent polyurethane foam cap.

Particularly suitable for tracheostomy tubes made of soft, sensitive plastic.

8. APPLICATION

Moisten the swab/sponge with clean water before use (see picture 1).

Remove the tracheostomy tube from the tracheostoma (see picture 2).

Clean the tracheostomy tube by slight forward/backward movements accompanied by turning movements (see picture 3).

9. HYGIENE INSTRUCTIONS

OPTIBRUSH® SWAB are intended for single patient use only and must not be cleaned or disinfected.

10. STORAGE

Store in a dry environment away from sunlight and/or heat.

11. SERVICE LIFE

Cleaning, disinfection or (re-)sterilisation as well as use longer than 24 hours can impair the safety and function of the product and are therefore not permissible!

12. DISPOSAL

The product must only be disposed of in accordance with the national regulations.

13. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or adverse events caused by unauthorised product alterations or improper use, care, and/or handling.

In particular, Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by modifications to the product or by repairs, if these modifications or repairs were not carried out by the manufacturer himself.

This applies – to the extent permitted by law – both to damages to the products themselves caused thereby as well as to any consequential damages caused thereby.

When using the product beyond the period of use specified under Section 11, and/or use, treatment, maintenance (cleaning, disinfection) or storage of the product in non-compliance with the specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects.

Should a serious incident occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, then this must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business (GTC) which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

The manufacturer reserves the right to make changes to the product at any time without notice.

OPTIBRUSH® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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