TRACHEOSAVE® PLACEHOLDER

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PICTROGRAM LEGEND

Pictogrammes listed below you'll find on the product packaging, if applicable.

	MRT suitable
	Catalogue number
	Batch code
	Use by
	Date of manufacture
	Content (in pieces)
	Consult instructions for use
	Keep away from sunlight
	Keep dry
	For reuse on a single patient
	Medical devices
	Sterilized using ethylene oxid
	Do not resterilize
	Do not use if package is damaged
	Single sterile barrier system with enclosed protective packaging
CE 0482	CE marking identification number of the notified body
	Manufacturer

TRACHEOSAVE® PLACEHOLDER

1. FOREWORD

These instructions for use are valid for **Tracheosave®** Placeholder.

These instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

When used for the first time, the products must be selected, used and inserted by appropriately trained physicians or medical professionals/medical device consultants.

Please read the instructions for use carefully before using the product for the first time!

Keep them in an easily accessible place for future reference.

Please keep the package for as long as you use the product.

It contains important information on the product!

2. SAFETY INFORMATION

When used for the first time, the products must only be selected needs-oriented, used and inserted by appropriately trained physicians or medical professionals.

We highly recommend keeping at least one replacement **Tracheosave®** Placeholder on hand to ensure a continuous supply.

Damaged **Tracheosave®** Placeholder must not be used and must be discarded immediately.

Tracheosave® Placeholder are single-patient products and intended for use by one patient only.

Do not re-use with another patient. Do not reprocess for re-use by another patient.

Never breathe continuously via the open **Tracheosave®** Placeholder - risk of obstruction!

In order to avoid the build-up of secretions, it is advisable to inhale and cough regularly.

If the shield is too loose and the **Tracheosave®** placeholder has become twisted or displaced in the tracheal lumen, this must be corrected.

If the breathing obstruction cannot be sufficiently removed in this way, the **Tracheosave®** placeholder must be removed.

In the event of problems with reinsertion (e.g. due to shrinkage of the stoma), the attending physician should be consulted immediately.

3. INTENDED PURPOSE

Tracheosave® placeholder is used to keep the tracheostoma open and ensures airway security.

The **Tracheosave®** can be closed for decannulation, blocking the airway through the tracheostoma.

4. INDICATION

The product is indicated in tracheotomy patients.

5. CONTRAINDICATION

Do not use in the case of respiratory bleeding, mandatory ventilation, untreated laryngotracheal stenosis and in patients with impaired consciousness and/or dependent persons in need of assistance.

Do not use if the patient is allergic to the material used.

6. COMPLICATIONS AND ADVERSE REACTIONS

Pressure sores, stenosis, necrosis, skin irritation, granulation tissue, cough irritation, bleeding and dysphagia.

If breathing is obstructed, first remove the stopper and aspirate.

Then check that the **Tracheosave®** Placeholder is correctly seated.

7. PRODUCT DESCRIPTION

The pack contains 1 **Tracheosave®** Placeholder, which is sterile packed and sterilized with ethylene oxide (EO).

The **Tracheosave®** placeholder is made of soft, mucosa-friendly silicone and consists of a half-tube (tracheal leg) adapted to the trachea and a tubular stoma leg to keep the tracheostoma open.

The non-removable shield can be locked in the required position on the stoma pocket.

This ensures that the **Tracheosave®** Placeholder cannot enter the airway during insertion.

The removable sealing plug is secured against loss and serves to seal the tracheostoma.

8. APPLICATION

Carefully examine the product to ensure that the packaging and contents have not been tampered with or damaged.

Do not use the product if the packaging has been damaged.

Check the use-by or expiry date.

Do not use after this date.

Carefully examine the **Tracheosave®** Placeholder before first use to make sure that it is not damaged and that there are no loose parts.

Should you notice any anomaly or anything unusual, DO NOT use the **Tracheosave®** Placeholder and return it to the manufacturer for inspection.

Before application, users should clean their hands.

The **Tracheosave®** Placeholder must always be cleaned and, if necessary, disinfected as follows before re-inserting according to the instructions provided below.

8.1 Perparing the Tracheosave® Placeholder

1. Pull the sealing plug out of the tubular tracheostomy tube before inserting the **Tracheosave®** Placeholder.
2. Push the insertion aid through the tracheostomy tube.

8.2 Inserting the Tracheosave® Placeholder

1. Using your thumb and index finger, fold the tracheal limb from the longer, caudal side upwards over the shorter, cranial side.
2. Fold the previously folded up leg a second time backwards over the middle towards the end of the insertion aid.
3. Gently push the insertion aid with the recess on the inside through the tubular tracheostomy tube onto the hand-fixed fold.
4. Fix the fold in the insertion aid.
5. Insert the **Tracheosave®** Placeholder into the tracheostoma with the folded longer leg aligned caudally.
6. Insert the longer limb (caudal) and the shorter limb (cranial) completely into the trachea.

7. Fix the **Tracheosave®** placeholder in position.
8. Remove the insertion aid.
9. The caudal tracheal limb unfolds noticeably.
10. Secure the **Tracheosave®** Placeholder by attaching a cannula support band to the shield of the **Tracheosave®** Placeholder.
Close the **Tracheosave®** Placeholder with the sealing plug if necessary.

8.3 Shorten the Tracheosave® Placeholder

The part of the **Tracheosave®** Placeholder in front of the shield and stopper can be carefully shortened to an individual length using atraumatic medical scissors.

WARNING!

An individual length adjustment may only be carried out by medically trained specialist personnel!

CAUTION!

Leave a safety distance of at least 1.5 cm in the direction of the sign!

8.4 Removal of the Tracheosave® Placeholder

1. Detach the cannula support band from the shield of the **Tracheosave®** placeholder.
2. Pull the **Tracheosave®** placeholder out of the tracheostoma with a short, strong downward pull.

9. HYGIENE INSTRUCTIONS

Tracheosave® placeholders must be cleaned/changed regularly according to the individual needs of the patient.

CAUTION!

Only clean the **Tracheosave®** Placeholder with cleaning agents approved by the manufacturer.

Never use aggressive household cleaners, detergents, high-concentration alcohol or denture cleaners.

This may be very hazardous to health.

In addition, the **Tracheosave®** Placeholder could be destroyed or damaged.

Only use cleaning products when the **Tracheosave®** Placeholder is outside the tracheostomy.

We recommend using the special cannula cleaning powder **OPTIBRUSH® CLEAN (REF 331110, REF 331100)** or **OPTICIT® Clean Set (REF 331180)** according to the manufacturer's instructions.

NOTE!

Be careful when cleaning the **Tracheosave®** Placeholder to avoid damage!

To make cleaning easier, we recommend using an **OPTIBRUSH® CONT (REF 31200)** cannula cleaning container with a sieve insert.

When doing this, hold the sieve insert at the top edge to avoid contact and contamination of the cleaning solution.

Connected aids must be removed before cleaning.

9.1 Cleaning

1. Rinse the **Tracheosave®** Placeholder thoroughly under running water.
2. Use a special cannula cleaning brush to remove stubborn and tough secretion residues.
3. The cleaning brush should only be used when the **Tracheosave®** placeholder has been removed and is already outside the tracheostoma.
4. Only use lukewarm water to prepare the cleaning solution and follow the instructions for use of the cleaning agent.
5. Add 1 level measuring spoons of **OPTIBRUSH® CLEAN** (REF 331110, REF 331100) cannula cleaning powder or a cleaning bag from the **OPTICIT®** Clean Set (REF 331180) to the cannula cleaning container.
6. Fill the cannula cleaning canister with water up to the lower edge of the lid thread (350 ml water) and gently swirl the cannula cleaning canister so that the powder begins to dissolve.
7. Immerse the sieve insert with the **Tracheosave®** in the prepared cleaning solution.
8. Gently rock the cannula cleaning can back and forth for 5 seconds.
9. After the immersion time of 15 minutes has elapsed, rinse the **Tracheosave®** thoroughly under running, lukewarm water.
10. There must be no cleaning agent residue on the **Tracheosave®**.
11. If necessary, e.g. B. if stubborn secretion residues cannot be removed with the cleaning bath, additional cleaning with a special cannula cleaning brush **OPTIBRUSH®** (REF 31850) or **OPTIBRUSH® PLUS** (with fiber attachment) (REF 31855) can help.
12. Rinse the **Tracheosave®** Placeholder thoroughly under running, lukewarm water.
13. After wet cleaning, the **Tracheosave®** Placeholder should be dried well with a clean, lint-free cloth.

9.2 Disinfection

ATTENTION!

Always clean according to the cleaning procedure described above before proceeding to disinfection (if applicable).

1. Open a disinfection bag from the **OPTICIT®** Clean Set (REF 331180), or take a measuring spoon of the **OPTICIT®** (REF 331170) disinfectant powder and empty the powder completely into the **OPTIBRUSH® CONT** (REF 31200) cannula cleaning container with sieve insert.
2. Fill the cannula cleaning canister with water up to the lower edge of the lid thread and gently swirl the cannula cleaning canister so that the powder begins to dissolve.
3. Place the **Tracheosave®** Placeholder in the solution.
4. Gently swirl the can back and forth for 5 seconds.
5. Then leave the **Tracheosave®** Placeholder in the disinfectant solution for 30 minutes.
6. After 30 minutes, rinse the **Tracheosave®** Placeholder thoroughly with running, lukewarm water.
7. Allow the Tracheosave Placeholder to dry and store properly.

9.3 Sterilisation

Resterilisation is not allowed.

10. STORAGE

Cleaned **Tracheosave®** placeholders that are not currently in use should be stored in a dry environment in a clean plastic container away from dust, sunlight and/or heat.

Tracheosave® placeholders that are still packaged sterile should be stored in a dry environment and protected from sunlight and/or heat.

11. SERVICE LIFE

The maximum period of use must not exceed 29 days.

The product must be disposed of after 60 cleaning cycles.

Damaged **Tracheosave®** Placeholders must be replaced immediately.

12. DISPOSAL

The product must only be disposed of in accordance with the national regulations.

13. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or adverse events caused by unauthorised product alterations or improper use, care, and/or handling.

In particular, Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by modifications to the product or by repairs, if these modifications or repairs were not carried out by the manufacturer himself.

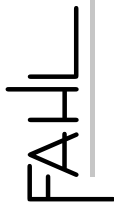
This applies – to the extent permitted by law – both to damages to the products themselves caused thereby as well as to any consequential damages caused thereby.

When using the product beyond the period of use specified under Section 11, and/or use, treatment, maintenance (cleaning, disinfection) or storage of the product in non-compliance with the specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects.

Should a serious incident occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, then this must be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business (GTC) which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

The manufacturer reserves the right to make changes to the product at any time without notice. **Tracheosave®** is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

CE 0482

Andreas Fahl

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August-Horch-Str. 4a

51149 Köln - Germany

Phone +49 (0) 22 03/29 80-0

Fax +49 (0) 22 03/29 80-100

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web www.fahl.com